

LIFELINKcentral™ AED Program Manager

Contents

- Instructions to log an inspection for one AED at a time
- Instructions to log an inspection for all AEDs at once

To log an inspection for one AED at a time:


1 Visually confirm your AED(s) are “OK” and the battery shows two or more bars

2 Log in

Log into LIFELINKcentral AED Program Manager.

3 Inspections

From “Equipment,” click “Manage Equipment” and then “Inspections” to see the inspection schedules for those AEDs for which you are responsible.

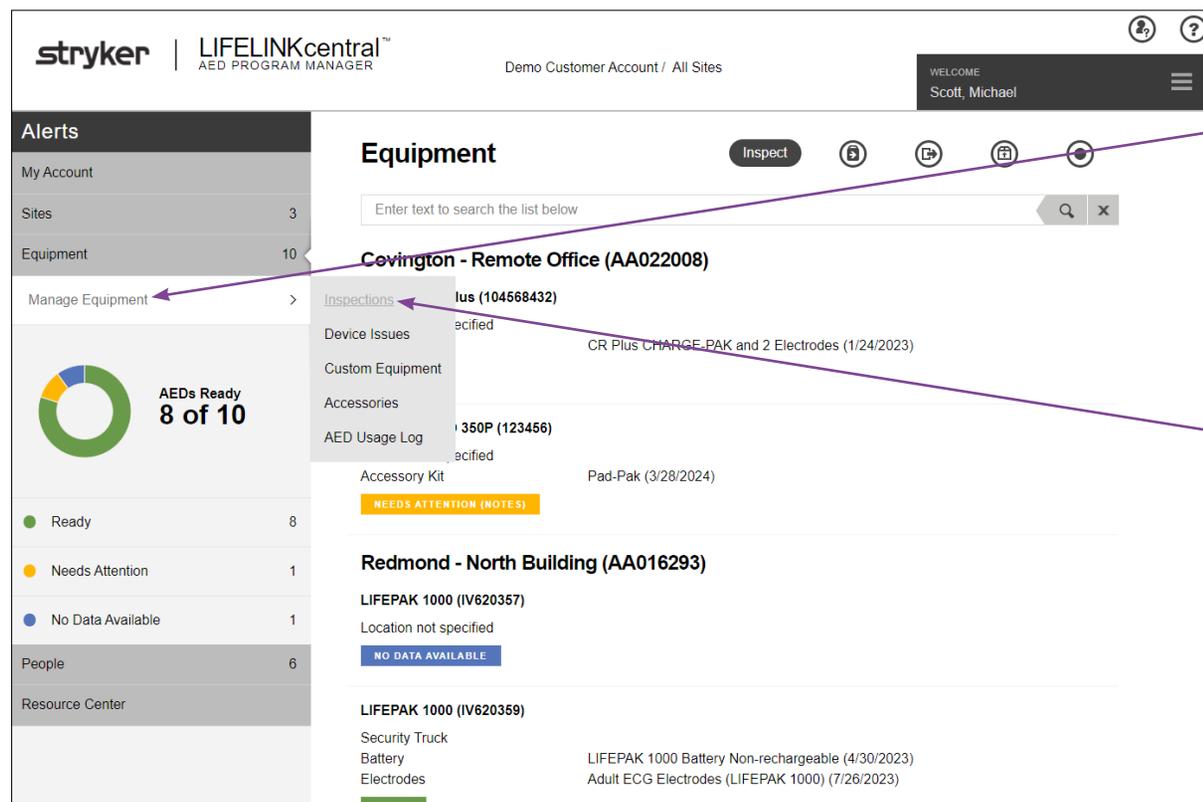

Manage Equipment

Inspections

4 Inspection Schedules

Next, click on the inspection schedule to which you would like to navigate. In this example, we will click on "Redmond-North Building."

 **Inspection Schedules**

Enter text to search the list below

Redmond - North Building (AA016293) ← **Navigate to Schedule**

AED Inspection Schedule

Person Responsible	Howard, Ryan
Frequency	Monthly
Last Inspection Date	12/28/2020

Redmond - South Building (AA016294)

AED Inspection Schedule

Person Responsible	Bernard, Andy
Frequency	Monthly
Last Inspection Date	12/28/2020

5 Select AED

Select the AED for which you are recording the inspection:

LIFEPAK 1000 (IV620357)

Performed By	System
Readiness Time	Not Specified
Location	Medical Clinic

NO DATA AVAILABLE

6 Verify information (Important for first inspection, optional for future inspections)

[Back to contents](#)

Expand each section and verify the information.

Perform Inspection ✕

LIFEPAK 1000 (IV620357)

EQUIPMENT INFORMATION ^

Location	Medical Clinic
Readiness Status	No Data Available

ACCESSORIES ∨

INSPECTION CHECKLIST ∨

NOTES ∨

Expand sections

7 Confirm AED readout

Proceed to Inspection Checklist and select the matching display readout:

EQUIPMENT INFORMATION

ACCESSORIES

INSPECTION CHECKLIST

Device Present

Visible Damage

Please select the set of indicators exactly as shown on the AED

<input checked="" type="radio"/>		OK	
<input type="radio"/>		OK	
<input type="radio"/>		OK	
<input type="radio"/>		OK	
<input type="radio"/>			

Readiness Status Ready

NOTES

Inspection Checklist

- a. Click the “Submit” button to submit your entries.
- b. The AED status will turn green for “Ready” confirming a successfully logged inspection:

LIFEPAK 1000 (IV620357)	
Performed By	Scott, Michael
Readiness Time	12/28/2020 9:17 AM
Location	Medical Clinic
READY	

You will be returned to the list view of the AED Inspection Schedule page. From there, repeat the steps above to perform a maintenance inspection on the next AED in the list.

To log an inspection for all AEDs at once:

8 Visually confirm the AEDs are “OK” and the battery on each shows two or more bars

9 Log In

Log into LIFELINKcentral AED Program Manager.

10 Inspections

From “Equipment,” click “Manage Equipment” and then “Inspections” to see the inspection schedules for those AEDs for which you are responsible.

The screenshot displays the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the Stryker logo, the product name 'LIFELINKcentral™ AED PROGRAM MANAGER', the user account 'Demo Customer Account / All Sites', and a welcome message for 'Scott, Michael'. A left sidebar contains navigation options: Alerts, My Account, Sites (3), Equipment (10), Manage Equipment, AEDs Ready (8 of 10), Ready (8), Needs Attention (1), No Data Available (1), People (6), and Resource Center. The main content area is titled 'Equipment' and features a search bar and a list of equipment items. Two callout boxes with purple backgrounds and white text are overlaid on the interface: 'Manage Equipment' points to the 'Manage Equipment' link in the sidebar, and 'Inspections' points to the 'Inspections' link in the dropdown menu for the 'Covington - Remote Office (AA022008)' equipment item.

Equipment Item	Status	Notes
Covington - Remote Office (AA022008)	Inspected	CR Plus CHARGE-PAK and 2 Electrodes (1/24/2023)
350P (123456)	Inspected	Accessory Kit Pad-Pak (3/28/2024)
Redmond - North Building (AA016293)	NO DATA AVAILABLE	LIFEPAK 1000 (IV620357) Location not specified
LIFEPAK 1000 (IV620359)	READY	Security Truck Battery LIFEPAK 1000 Battery Non-rechargeable (4/30/2023) Electrodes Adult ECG Electrodes (LIFEPAK 1000) (7/26/2023)

11 Inspection Schedules

[Back to contents](#)

Next, click on the Inspection Schedule to which you would like to navigate. In this example, we will click on “Redmond-North Building” to log an inspection for all AEDs affiliated with that schedule:

 **Inspection Schedules**

Enter text to search the list below

Redmond - North Building (AA016293)  **Navigate to Schedule**

AED Inspection Schedule

Person Responsible	Howard, Ryan
Frequency	Monthly
Last Inspection Date	12/28/2020

Redmond - South Building (AA016294)

AED Inspection Schedule

Person Responsible	Bernard, Andy
Frequency	Monthly
Last Inspection Date	12/28/2020

12 AED Inspection Schedule

Once you are on the landing page for the “Redmond-North Building” Inspection Schedule, click the “Inspect All” button to log a check for all AEDs at once.

The screenshot shows the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the Stryker logo, the user's name (Scott, Michael), and the account information (Demo Customer Account / Redmond - North Building). The main content area is titled "AED Inspection Schedule" and features a search bar, a "Complete Inspection" button, and a list of AEDs. A purple callout box labeled "Inspect All" points to the "Inspect All" button in the top right corner of the main content area.

Equipment	Performed By	Readiness Time	Location	Status
LIFELINKcentral™ AED PROGRAM MANAGER	System	Not Specified	Security Truck	NO DATA AVAILABLE
LIFELINKcentral™ AED PROGRAM MANAGER	Majumdar, Shalini	12/28/2020 8:52 AM	North Building, Lobby	READY
LIFELINKcentral™ AED PROGRAM MANAGER	Majumdar, Shalini	12/28/2020 8:52 AM	First floor, next to central stairwell	READY
LIFELINKcentral™ AED PROGRAM MANAGER	Scott, Michael	12/28/2020 9:17 AM	Medical Clinic	

13 Confirm

The following prompt will appear, asking you to confirm you would like to update the readiness status of all AEDs associated with this schedule to “Ready.” Click “Yes” to complete the check:

The screenshot shows a "Complete Inspection" dialog box. The text inside the dialog reads: "The readiness status of all Equipment not previously recorded today will be set to 'Ready' with today's date. Do you want to proceed?". There are two buttons: "Yes" and "No". A purple callout box labeled "Click Yes" points to the "Yes" button.

14 Completed

[Back to contents](#)

You will be routed to the list view for all AEDs on that Inspection Schedule. The maintenance inspection for all AEDs listed in this schedule is complete, and all AEDs will show green for "Ready" with today's date; in this example, 12/28/2020 at 9:23 AM:

LIFEPAK 1000 (IV620357)	
Performed By	Scott, Michael
Readiness Time	12/28/2020 9:23 AM
Location	Medical Clinic
READY	
<hr/>	
LIFEPAK 1000 (IV620359)	
Performed By	Scott, Michael
Readiness Time	12/28/2020 9:23 AM
Location	Security Truck
READY	

For further information, please contact Stryker at 800 442 1142 (U.S.), 800 668 8323 (Canada) or visit our website at strykeremergencycare.com

Emergency Care

Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: HeartSine, LIFELINKcentral, Stryker. All other trademarks are trademarks of their respective owners or holders.

The absence of a product, feature, or service name, or logo from this list does not constitute a waiver of Stryker's trademark or other intellectual property rights concerning that name or logo.

GDR 3336346_C

Copyright © 2021 Stryker

Distributed by:

Physio-Control, Inc.
11811 Willows Road NE
Redmond, WA, 98052 U.S.A.
Toll free 800 442 1142
strykeremergencycare.com

Distributed in Canada by:

Stryker Canada
2 Medicorum Place
Waterdown, Ontario
L8B 1W2
Canada
Toll free 800 668 8323