

LIFEPAK® CR2 AED manual maintenance inspection update LIFELINKcentral™ AED Program Manager

Contents

- Instructions to log an inspection for one AED at a time.
- Instructions to log an inspection for all AEDs at once.

To log an inspection for one AED at a time:

1. Visually confirm your AED(s) flashing green indicator.
2. Log in
 - a. Log into LIFELINKcentral AED Program Manager.
3. Select Site
 - a. From “Sites” select the site for which you would like to log the inspection (Redmond – North Building).

The screenshot shows the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the Physio Control logo, the user name 'Riggs, Malorie', and the account type 'Demonstration Customer Account / All Sites'. The left sidebar shows a navigation menu with 'Alerts', 'My Account', 'Sites' (2), 'Needs Attention' (2), 'Equipment' (4), 'People' (5), and 'Resource Center'. The main content area is titled 'Sites' and features a search bar with the text 'Enter text to search the list below'. Below the search bar, two site entries are listed:

- Redmond-North Building (SS774426)**
 - Address: 11811 Willows Road NE, Redmond, Washington, 98052
 - Primary Site Coordinator: Riggs, Malorie
 - Phone Number: 425-867-4700
 - E-mail: mriggs@sample.com
 - NEEDS ATTENTION (DEVICE)
- Redmond-South Building (SS776691)**
 - Address: 11811 Willows Road NE, Redmond, Washington, 98052
 - Primary Site Coordinator: Graubard, Angela
 - Phone Number: 425-867-4700
 - E-mail: agraubard@sample.com
 - NEEDS ATTENTION (DEVICE)

4. Click the Inspect button to record an inspection.

The screenshot displays the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the Physio Control logo, the text 'LIFELINKcentral™ AED PROGRAM MANAGER', the user account 'Demonstration Customer Account / Redmond-North Building', and a welcome message 'WELCOME Riggs, Malorie'. A left sidebar contains navigation options: Alerts, My Account, Sites (2), Equipment (2), People (4), and Resource Center. The main content area shows the 'Site' details for 'Redmond-North Building (SS774426)'. A yellow box highlights the 'Inspect' button. Below the site title, a list of site details is provided:

Account	Demonstration Customer Account (AETT0901)
Primary Site Coordinator	Riggs, Malorie
Site Phone Number	425-867-4700
Create Date	7/15/2016
Address	11811 Willows Road NE Redmond Washington 98052 United States
Local EMS Agency	Not Specified
AED Post-Event Report Recipient	Dames, M.D., Joanne
Required number of Trained Responders per Site	1

5. Select AED
 - a. Select the AED for which you are recording the inspection.

LIFEPAK CR2 (48248214)

Performed By	System
Readiness Time	Not Specified
Location	First Floor Lobby

NO DATA AVAILABLE

6. Verify information (important for first inspection, optional for future inspections).
 - a. Expand each section and verify the information.

Perform Inspection

LIFEPAK CR2 (48248214)

EQUIPMENT INFORMATION

ACCESSORIES

INSPECTION CHECKLIST

Device Present

Visible Damage

Readiness Indicator Flashing Off

Override Status

Readiness Status Ready

NOTES

Submit Cancel

7. Confirm AED readout
 - a. Proceed to Inspection Checklist and confirm the flashing readiness indicator is displayed.

Perform Inspection ✕
LIFEPAK CR2 (48248214)

EQUIPMENT INFORMATION ∨

ACCESSORIES ∨

INSPECTION CHECKLIST ∧

Device Present

Visible Damage

Readiness Indicator Flashing Off

Override Status

Readiness Status Ready

NOTES ∨

Submit Cancel

- b. Click the “Submit” button to submit your entry.
 - c. The AED status will turn green for “Ready”, confirming a successfully logged inspection:

LIFEPAK CR2 (48248214)

Performed By Riggs, Malorie

Readiness Time 4/29/2020 2:41 PM

Location First Floor Lobby

READY

You will return to the list view of the AED Inspection Schedule page. From there, repeat the steps above to record a maintenance inspection for the next AED in the list.

To log an inspection for all AEDs at once:

8. Visually confirm the AEDs display the flashing green readiness indicator.
9. Log in
 - a. Log into LIFELINKcentral AED Program Manager.
10. Inspections
 - a. From “Equipment,” click “Manage Equipment”, and then click “Inspections” to see the inspection schedules for those AEDs for which you are responsible.

The screenshot shows the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the Physio Control logo, the product name 'LIFELINKcentral™ AED PROGRAM MANAGER', and the user account information 'Demonstration Customer Account / All Sites' and 'WELCOME Riggs, Malorie'. The left sidebar contains navigation options: Alerts, My Account, Sites (2), Equipment (4), Manage Equipment, and Resource Center. A donut chart in the sidebar shows 'AEDs Ready 3 of 4'. The main content area is titled 'Equipment' and features an 'Inspect' button and a search bar. A dropdown menu is open over the 'Equipment' sidebar item, with 'Inspections' highlighted. The main content area displays a list of equipment for two buildings: Redmond-North Building (SS774426) and Redmond-South Building (SS776691). The Redmond-North Building list includes 'LIFELINKcentral CR2 (48248214)' and 'LIFELINKcentral CR2 (48248216)'. The Redmond-South Building list includes 'LIFELINKcentral CR2 (47987626)'. A 'NO DATA AVAILABLE' button is visible under the Redmond-South Building section.

11. Inspection Schedules

- a. Next, click on the inspection schedule to which you would like to navigate. In this example, we will click on "Redmond – South Building," to log an inspection for all AEDs affiliated with that schedule:

The screenshot displays the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the Physio Control logo, the text "LIFELINKcentral™ AED PROGRAM MANAGER", the user account "Demonstration Customer Account / All Sites", and a welcome message for "Riggs, Malorie".

The left sidebar contains a navigation menu with the following items:

- Alerts
- My Account
- Sites (2)
- Equipment (4)
- Manage Equipment (>)

Below the menu is a donut chart showing "AEDs Ready 3 of 4". The chart is divided into a green segment (3) and a blue segment (1). A legend below the chart indicates:

- Ready (Green circle) 3
- No Data Available (Blue circle) 1

The main content area is titled "Inspection Schedules" and features a search bar with the placeholder text "Enter text to search the list below".

Two inspection schedule entries are listed:

- Redmond-North Building (SS774426)**
 - AED Inspection Schedule**
 - Person Responsible: Riggs, Malorie
 - Frequency: Monthly
 - Last Inspection Date: 4/29/2020
- Redmond-South Building (SS776691)**
 - AED Inspection Schedule**
 - Person Responsible: Riggs, Malorie
 - Frequency: Monthly
 - Last Inspection Date: 4/3/2020

12. AED Inspection Schedule

- a. Once you are on the landing page for the “Redmond – South Building” inspection schedule, click “Inspect All”.

The screenshot displays the LIFELINKcentral AED Program Manager interface. The top navigation bar includes the Physio Control logo, the text "LIFELINKcentral™ AED PROGRAM MANAGER", the user account information "Demonstration Customer Account / Redmond-South Building", and a welcome message "WELCOME Riggs, Malorie".

The left sidebar contains a navigation menu with the following items:

- Alerts
- My Account
- Sites (2)
- Equipment (2)
- Manage Equipment (>)
- AEDs Ready 2 of 2 (indicated by a green donut chart)
- Ready (2)
- People (1)

The main content area is titled "AED Inspection Schedule" and features a search bar with the placeholder text "Enter text to search the list below". A yellow box highlights the "Inspect All" button. Below the search bar, two AED units are listed:

LIFEPAK CR2 (48125068)	
Performed By	System
Readiness Time	4/3/2020 1:57 AM
Location	First Floor Lobby
READY	

LIFEPAK CR2 (47987626)	
Performed By	System
Readiness Time	4/9/2020 8:47 AM
Location	Warehouse
READY	


13. Confirm

- a. The following prompt will appear, asking you to confirm if you would like to update the readiness status of all AEDs associated with this schedule to "Ready". Click "Yes" to complete the check:



14. Completed

- a. You will be routed to the list view for all AEDs on that inspection schedule. The maintenance inspection for all AEDs under this schedule is complete, and all AEDs will show green for "Ready" with today's date.

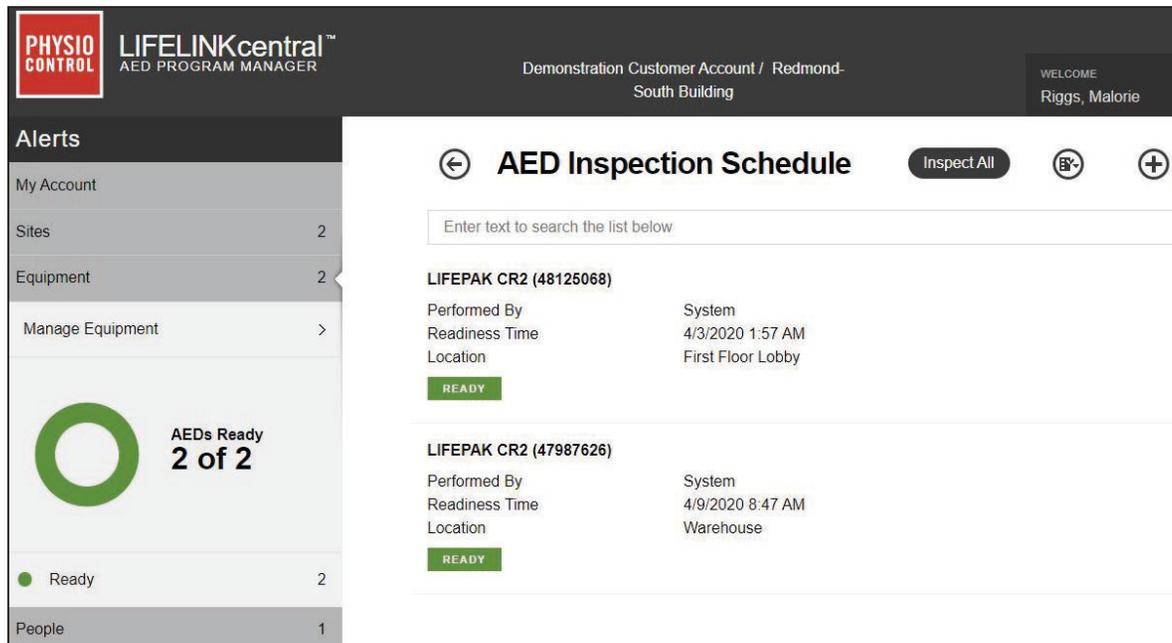

For further information, please contact your Account Support Representative or call 888-627-0007