



LIFEPAK® 1000 AED

User's Checklist

Year:

Unit Serial Number:

Location:

MONTH
DATE
INITIALS

Instruction and Recommended Corrective Action

Check defibrillator/electrode package, wires & connector for damage or foreign substances.

 is visible on readiness display. If not, refer to Troubleshooting Table. Note level of battery charge. If one or no bars are visible refer to troubleshooting table.

Note use by date on electrode pads and battery.
Electrodes: _____ Battery: _____

Extra set of electrode pads stored with AED (optional).
Use by date: _____

Infant/Child electrode pads* stored with AED (optional).
Use by date: _____

Resuscitation Kit containing: disposable gloves, face mask, trauma scissors and razor stored with AED.

Other resuscitation equipment stored with AED.

* The LIFEPAK 1000 AED may be used on children up to 8 years old or 25kg (55 lbs) only with the Infant/Child Reduced Energy Defibrillation Electrodes.

Troubleshooting Readiness Display

	Service needed	Attempt to use AED in an emergency. If AED is inoperable, retrieve another AED if possible and continue CPR until EMS arrives.
	Low Battery Indicator	Replace battery.
	Extremely Low Battery Indicator	Replace battery immediately.

NOTE: If corrective action is needed leave box unchecked, complete troubleshooting log, and notify: _____.

